



## HOW-TO-GUIDE FOR CREATING A NEGATIVE PRESSURE AMBULANCE



- **TURN OFF HVAC**
- Measure length needed and cut
- Ensure section cut will cover entire compartment opening
- Starting at top tape off compartment forming a seal around the compartment pass through
- Be sure to avoid taping over or covering up the vents to the electrical panel
- Tape all 4 sides including the floor to create a seal

### **TURN ON EXHAUST VENT TO HIGH HVAC should be OFF during transport.**

- Close doors of compartment
- Confirm negative pressure seal by observing plastic pull towards vent

**Cover remainder of ambulance patient compartment sides- no need for ceiling or floor.**

